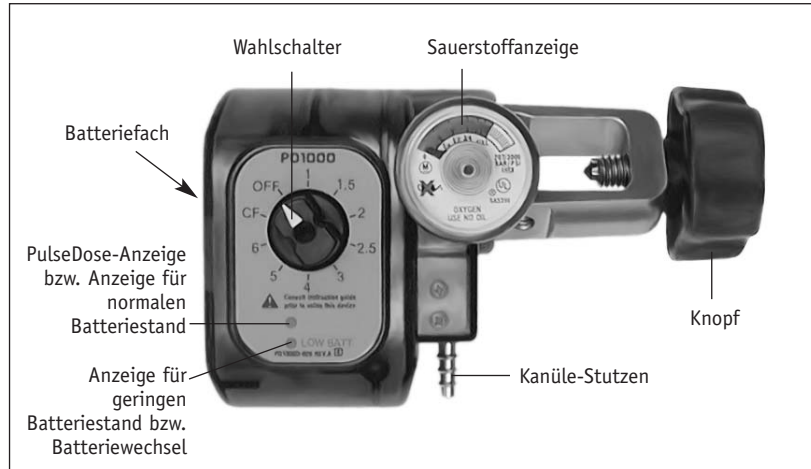
## DEUTSCH

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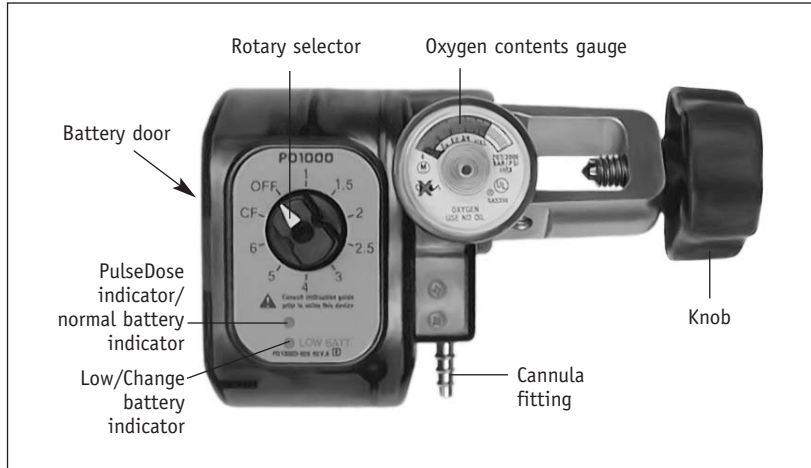
*DeVilbiss® PD1000 PulseDose® LT Compact Quick Reference Card*

**USING YOUR PULSEDOSÉ COMPACT CONSERVING DEVICE**

1. Install batteries and verify battery level status.
2. Attach the conserving device to the oxygen cylinder.
3. Open the cylinder.
4. Turn the rotary selector to the correct prescription flow setting.
5. Attach the standard nasal cannula to the conserving device and to your nose and face.
6. Breathe normally, the conserving device will deliver a bolus of oxygen at the leading edge of inspiration on every breath up to 40 BPM.
7. When you are finished using the PulseDose Compact Conserving Device, turn your cylinder to the closed position and rotate the rotary selector to the "OFF" position.

**NOTE-** Further explanation found in instruction guide.

DeVilbiss Healthcare LLC  
 100 DeVilbiss Drive  
 Somerset, PA 15501-2125  
 USA  
 800-338-1988  
 814-443-4881  
 www.DeVilbissHealthcare.com

*DeVilbiss® PD1000 PulseDose® LT Compact Quick Reference Card*

*When you have a problem with the operation of your DeVilbiss PulseDose Compact Conserving Device, please take a few moments to check for these possible causes before calling your home medical equipment provider.*

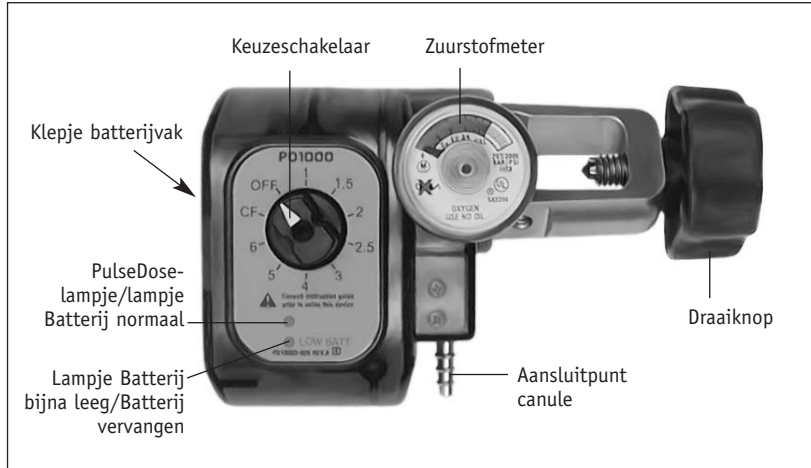
**PROBLEM- UNIT WON'T PULSE**

- Be sure the rotary selector is set to the appropriate flow setting.
- Check the battery level by turning the unit off, waiting five seconds, then turning the knob to the number one setting. The green "Normal Battery Indicator" should illuminate. If the "Low Battery" light illuminates, change battery.
- Take cannula off nose and ensure tubing isn't kinked.
- Attach cannula to nose and breathe.
- What does the pressure gauge read?
  - Empty- Change tank.
  - Not Open- Be sure the tank valve is all the way open. This is especially important on tanks that are close to empty.
- If unit still does not pulse, call your HME provider.

**NOTE-** When changing batteries, first turn the rotary selector to the "OFF" position.

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